

Tetric® plus

1

Self-Etch



or

Etch & Rinse



15-30 s



10-15 s

or

Selective Enamel-Etch



15-30 s



2

Adhese® Universal
VivaPen®



2-3 clicks



20 s



AIR



3

Tetric® plus
Flow



Optional

As initial layer



or

up to 4 mm

Small restorations without a capping layer



4

Tetric® plus
Fill



up to 4 mm



Bluephase®
PowerCure



For permanent teeth only
For deciduous teeth see instructions for use

	Tetric plus
500 – 900 mW/cm ²	20 s
1,000 – 1,300 mW/cm ²	10 s
1,800 – 2,200 mW/cm ²	5 s
2,700 – 3,300 mW/cm ²	3 s

Date information prepared: 2023-10-12
762305 / Rev. 0

ivoclar

see
Instructions
www.ivoclar.com/e/ifu

Rx ONLY
For dental use only!

CE 0123

Tetric® plus

Tetric plus



VITA*
classical shades

A1, A2,
B2, C1, D2

A3, C2,
C3, D3, D4

A3.5, A4,
B3, B4, C4

B1,
Bleach

* no registered trademark of Ivoclar



Manufacturer: Ivoclar Vivadent AG
Bendererstrasse 2, 9494 Schaan/Liechtenstein

www.ivoclar.com

ivoclar